



RX Herculink Elite

Peripheral Stent System

The deliverability of
RX Herculink

The performance of CoCr

RX Herculink Elite

Peripheral Stent System


Building on the proven deliverability of RX Herculink, using the advanced metallurgy of cobalt chromium, RX Herculink Elite sets standards in renal stenting: Excellent radiopacity, low profile, high radial strength and superb stent flexibility.



Excellent radiopacity





Superb flexibility

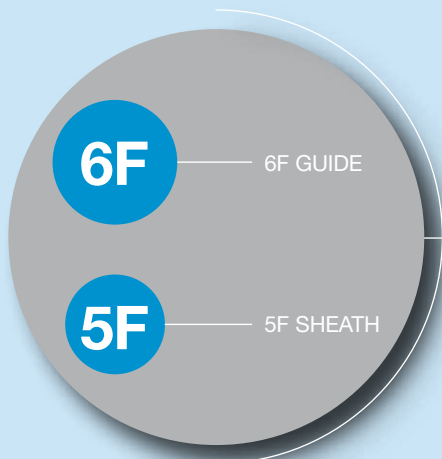

Low profile



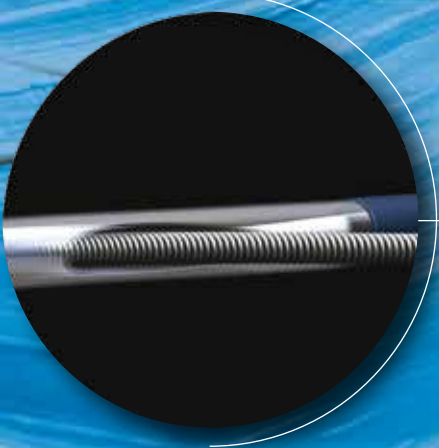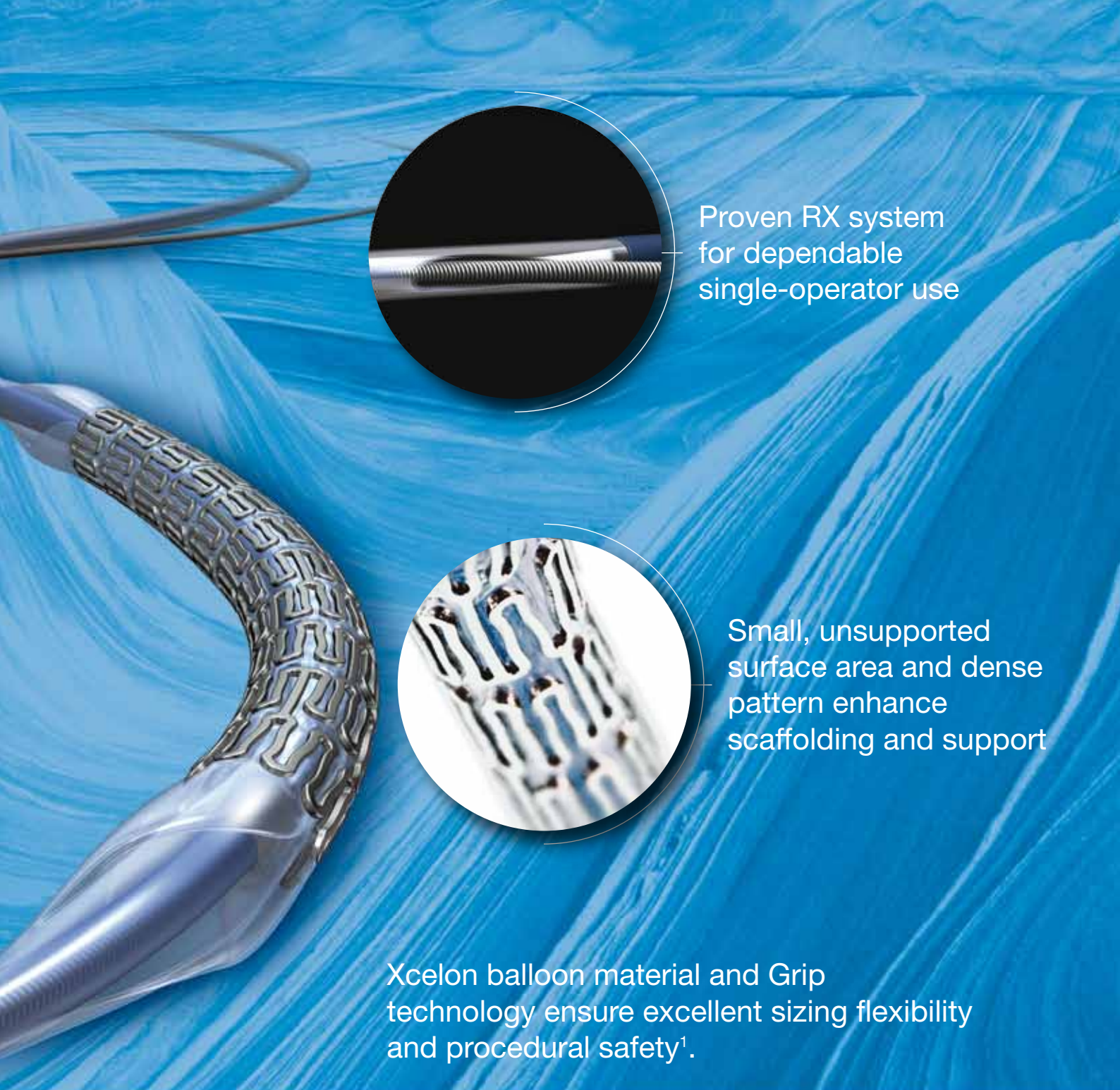


Proven RX system for dependable single-operator use



Small, unsupported surface area and dense pattern enhance scaffolding and support

Xcelon balloon material and Grip technology ensure excellent sizing flexibility and procedural safety¹.

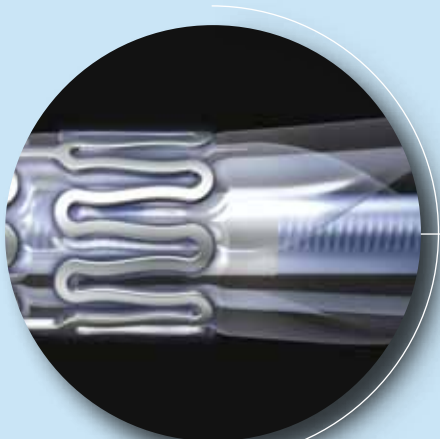

Grip technology helps ensure excellent stent retention, smooth surface transitions and protection of stent edges

14 atm RBP delivers high-pressure capabilities on a low-profile system

RX Herculink Elite

Peripheral Stent System

.014

STOCK NUMBER		PREFIX CATHETER LENGTH (cm)	SUFFIX STENT LENGTH (mm)	STENT DIAMETER (mm)	MAXIMUM POST DILATATION DIAMETER (mm)	RATED BURST PRESSURE (atm)	SHAFT DIAMETER PROXIMAL / DISTAL (F)	MINIMUM SHEATH SIZE (F)	MINIMUM GUIDING CATHETER ID (F / in)
80	135								
1011521	1011522	-12	4.0	7.0			3.3F / 2.9F		
1011524	1011525	-12	4.5	7.0			3.3F / 2.9F		
1011527	1011528	-12	5.0	7.0			3.3F / 2.9F		
1011530	1011531	-12	5.5	7.0			3.3F / 3.3F		
1011533	1011534	-12	6.0	7.0			3.6F / 3.3F		
1011536	1011537	-12	6.5	8.0			3.6F / 3.5F		
1011521	1011522	-15, -18	4.0	7.0	14		3.3F / 2.9F	5F	6F (.067")
1011524	1011525	-15, -18	4.5	7.0			3.3F / 2.9F		
1011527	1011528	-15, -18	5.0	7.0			3.3F / 2.9F		
1011530	1011531	-15, -18	5.5	7.0			3.3F / 3.3F		
1011533	1011534	-15, -18	6.0	7.0			3.6F / 3.3F		
1011536	1011537	-15, -18	6.5	8.0			3.6F / 3.5F		
1011539	1011540	-15, -18	7.0	8.0			3.6F / 3.5F		

PRESSURE (atm)	STENT DIAMETER (mm)						
	4.0	4.5	5.0	5.5	6.0	6.5	7.0
11	4.00	4.50	5.00	5.50	6.00	6.50	7.00
12	4.08	4.60	5.13	5.61	6.13	6.61	7.14
13	4.15	4.68	5.25	5.72	6.25	6.71	7.27
14 (RBP)	4.21	4.77	5.35	5.82	6.36	6.81	7.39
15	4.28	4.84	5.46	5.91	6.47	6.90	7.50
16	4.33	4.91	5.55	5.99	6.56	6.98	7.61
17	4.39	4.98	5.64	6.07	6.66	7.06	7.70

- NOMINAL
- RATED BURST PRESSURE

Abbott Vascular Australia

Level 8, 15 Talavera Road
North Ryde NSW 2113
Tel: 1800 550 939
Tel: +61 2 8879 2800
Fax: +61 2 8879 2805

Abbott Vascular New Zealand

Level 3, 6 Kingdon Street
New Market Auckland
Tel: 0800 827 285
Tel: +64 524 6580
Fax: +64 9524 6584

1. Data on file at Abbott Vascular.

This product is intended for use by or under the direction of a physician. Prior to use, it is important to read the package insert thoroughly for instructions for use, warnings and potential complications associated with the use of this device.

Information contained herein for distribution in Australia and New Zealand only.

For more information, visit our web site at abbottvascular.com